

Product Name: Gen. 3 Breast Pump Flange**Material:** Medical-Grade Silicone**Dimensions:** 71 x 71 x 103mm**Product Description:**

The Haakaa Silicone Breast Pump Flange attachment is fully compatible with all Gen. 3 Silicone Bottles, transforming them into our famous Silicone Breast Pump!

We have kept the same simple and effective design that you know and love, except now you can express directly into a detachable silicone bottle for easy feeding or storage! With a quick twist of the top, you can swap out your breast pump flange for any of our range of optional interchangeable attachments (each sold separately). We have a baby bottle lid to feed your little one expressed milk, a feeding spoon dispenser for when your baby moves on to solids, a sippy bottle top to help your independent toddler transition from a bottle to a cup and sealing disks to preserve any stored food or milk.

Made from premium medical-grade silicone, this compact pump flange is perfect for long-distance plane or car rides, or simply anywhere you would like to express quickly and discreetly.

Features:

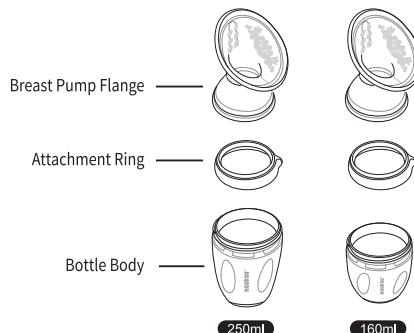
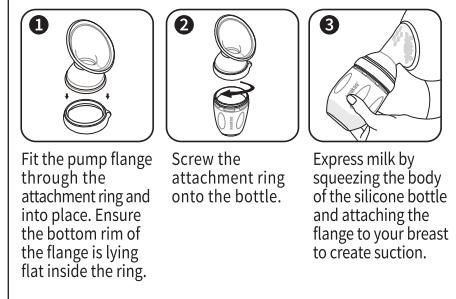
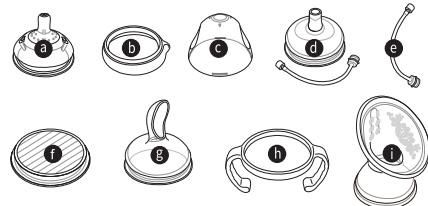
- Express breastmilk with the power of continuous suction!
- Made of premium medical-grade silicone.
- Express breastmilk directly into the bottle.
- Compatible with all Gen. 3 silicone bottles.
- Accommodates different breast sizes.
- No cords or batteries required.
- Small, portable and convenient.
- Safe in microwaves, boiling water and steam sterilisers.
- BPA, PVC and phthalate-free.

Cleaning and Care:

Wash before and after each use. Although dishwasher safe, we recommend hand washing in warm soapy water and rinsing and drying thoroughly. Do not use any bleach-based cleaners or tablets to sterilise or clean your Haakaa products. This product can be sterilised with any steam sterilising system or by boiling in water for 2-3 minutes.

Caution:

- Check product condition regularly. Replace if this product shows any signs of damage.
- Do not store near any sharp objects.
- Only soft bristle brushes or soft sponges should be used to clean this product. Hard scourers may scratch the surface.
- This product is not a toy. Do not use this product for anything other than its intended use.
- Store product in a cool dry place and avoid direct sunlight.
- Adult supervision is required.

Silicone Baby Bottle Diagram**Assembly Instructions****Optional Interchangeable Attachments**

Haakaa Silicone Baby Bottles are compatible with all Gen. 3 Bottle Series attachments.

○ a) Anti-Colic Nipple	○ f) Sealing Disk
○ b) Attachment Ring	○ g) Silicone Bottle Feeding Spoon Head
○ c) Replacement Cap	○ h) Silicone Bottle Handle
○ d) Sippy Spout	○ i) Silicone Bottle Breast Pump Flange
○ e) Automatic Straw - S/L	

PLEASE NOTE: Optional interchangeable attachments each sold separately. This product is an attachment for our Gen. 3 Silicone Bottles, and will not work by itself.

Volume measurements are an approximation and should be used for general reference only.