

haakaa®

Our New Mum Starter Pack is the perfect way to get you started on your breastfeeding journey and it also makes the perfect gift for a new mum! This set includes all your Haakaa mama favourites, including our one-piece silicone breast pump to help you express milk, a stopper and cap to prevent spills and nursing pads to keep you comfortably dry when you're not pumping.

How to Pump On-the-Go



1. Squeeze the pump base, centre over your nipple and release. This will draw your milk with continuous suction.




2. The Silicone Flower Stopper sits tightly in the neck of your breast pump to prevent any spills.



3. The Silicone Cap keeps any milk in your pump safe. It also keeps your pump hygienic and dust-free after cleaning.

Wearing Your Nursing Pads

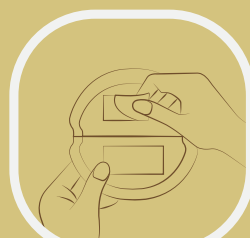

1. Pull the paper off the anti-slip adhesive.



2. Place the nursing pad inside your bra with the adhesive side facing away from your body.



3. Once full, dispose and replace the pad as required.

Please note: Always follow your country's guidelines and recommendations to safely express and store breast milk.

More detailed instructions are inside the box.

haakaa® IS A REGISTERED TRADEMARK OF THINK GREEN LTD, NEW ZEALAND, 241 Bush Road, Rosedale, Auckland. Designed in New Zealand. Made in China. For more information, please visit www.haakaa.co.nz



haakaa®

New Mum Starter Pack

NON-TOXIC • NO BISHENOL A • NO PHthalATES • BPA FREE



1 Set

This set contains:




1 x Gen. 2 Silicone Breast Pump
Material: 100% Silicone



1 x Silicone Flower Stopper
Material: 100% Silicone



1 x Silicone Breast Pump Cap
Material: 100% Silicone

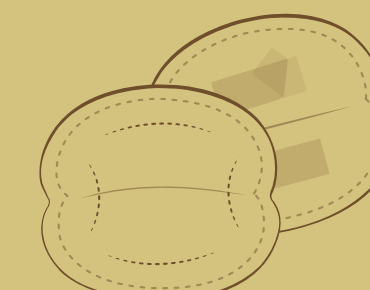

36 x Disposable Nursing Pads
Materials: Wood Pulp, Cotton, Tissue Paper, Nonwoven, PE Film, SAP