

DISPOSABLE NURSING PADS

PRODUCT MANUAL English

Product Name: Disposable Nursing Pads - Butterfly Shape

Materials: Wood Pulp Cotton, Nonwoven Fabric, Tissue Paper,
PE Breathable Film, Super Absorbent Polymer, Absorbent Cotton

Dimensions: 120 x 110mm

Contents: 36 x Disposable Nursing Pads

Product Description:

Haakaa Disposable Nursing Pads are designed with a special absorbent polymer and stay dry lining to draw milk away from the skin and into the core of the pad. They're low profile, giving you the confidence to wear them during the day without worrying about lumps and bumps under your clothes. Our Disposable Nursing Pads keep you and your clothing dry to help you to avoid those embarrassing milk stains!

Features:

- Quickly draws moisture away from the skin.
- Ultra-soft and comfortable to wear.
- Slim and lightweight with a discreet low profile design.
- Made with natural materials.


Cleaning and Care:

Dispose of each pad responsibly after use.


Caution:

Do not reuse. If not disposed of immediately, the pads may begin to smell due to the expiry of your breast milk. Change your pads when damp to avoid constant moisture around the nipple, which can foster the growth of bacteria. Make sure your nipple is dry before applying a clean pad. If your nipples are irritated, apply nipple cream before the breast pad.

Instructions:

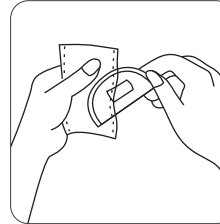


- 1 Remove the nursing pad from its packaging.

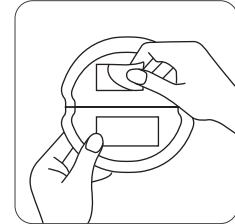

- 2 Pull the paper off the anti-slip adhesive.



- 3 Place the nursing pad inside your bra with the adhesive side facing away from your body.

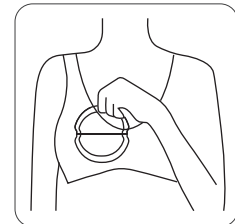

- 4 Once full, dispose and replace the pad as required.