

The Haakaa logo consists of the word "haakaa" in a white, lowercase, sans-serif font, positioned inside a dark grey square.

SILICONE BREAST PUMP CAP

PRODUCT MANUAL

English

Product Name: Silicone Breast Pump Cap

Material: 100% Food Grade Silicone

Dimensions: 85 x 85 x 12mm

Colour: Grey

Product Description:

Keep your Haakaa Breast Pump dust-free between uses and your breast milk safe from airborne nasties with the Haakaa Silicone Breast Pump Cap! It is compatible with all of our breast pumps to prevent spills, and helps to keep them clean and hygienic between uses!

This cap is made of 100% food grade silicone and fits securely over the top of your pump's flange to keep it free of airborne dust and other nasties. For an extra-secure seal, place the cap on your pump with the valve open and squeeze your pump very gently before closing the valve again.

Features:

- Seals your pump to keep it clean and hygienic.
- Leakproof design to prevent spills.
- Compatible with all Haakaa Silicone Breast Pumps.
- Valve on cap helps to create an extra-tight seal.
- Made of 100% food grade silicone.
- Safe in dishwashers and boiling water.
- BPA, PVC and phthalate-free.

Cleaning and Care:

Remove cap from pump and open valve before sterilising. Wash before and after each use. Although dishwasher safe, we recommend hand washing in warm soapy water and rinsing and drying thoroughly. Do not use any bleach based cleaners or tablets to sterilise or clean your Haakaa products. To sterilise, use a steam steriliser or boil in water for 2-3 minutes.

Caution:

- ⊙ Check product condition regularly. Replace if product shows any signs of damage.
- ⊙ Do not store near any sharp objects.
- ⊙ Only soft bristle brushes or soft sponges should be used to clean this product. Hard scourers may scratch the surface.
- ⊙ This product is not a toy. Do not use for any other purpose.
- ⊙ Store product in a cool dry place and avoid direct sunlight.