

The Haakaa logo consists of the word "haakaa" in a white, lowercase, sans-serif font, positioned inside a dark grey square.

SILICONE BREAST PUMP STRAP

PRODUCT MANUAL

English

Product Name: Silicone Breast Pump Strap

Material: Food Grade Silicone

Dimensions: 1130 x 17 x 6 mm

Colours: Bluestone/Blush/Rust/Suva Grey

Product Description:

Protect your breast pump from being accidentally knocked off with Haakaa's Silicone Breast Pump Strap! Our adjustable strap is made of 100% food grade silicone and hangs around your neck while you pump to keep that precious liquid gold safe. No more worrying about kicky babies or your breast pump falling off!

Features:

- Prevents your pump from falling to the ground.
- Made of 100% food grade silicone.
- One-piece design.
- Adjustable length.
- Portable and convenient.
- BPA, PVC and phthalate-free.

Instructions:

1. Insert your breast pump through the loop at the end of the strap, ensuring that it fits tightly and securely around the neck of the pump.
2. Holding the beaded end of the strap, insert the end through the opening next to the loop.
3. Pull the beaded end through the opening until the neck strap reaches the desired length.
4. Hang around your neck.

Cleaning and Care:

Wash after each use. Although dishwasher safe, we strongly recommend sterilising using a steam steriliser or by boiling in water for 2-3 minutes. Do not use any bleach-based cleaners or tablets to clean or sterilise this product.

Caution:

Check product condition regularly - replace if this product shows any signs of damage. This product is not a toy. Do not give to babies as it poses a choking hazard. Please do not use this product for anything other than its intended use.