

# SILICONE MICROWAVE STERILISER BAG

## Product Manual

English

**Product Name:** Silicone Microwave Steriliser Bag

**Material:** Food-Grade Silicone

**Dimensions:** 250 x 76 x 190mm

**Volume:** 50 oz./1500ml

**Colour:** Grey

**Quantities:** 1pc/2pcs

### Product Description:

Who says you need big, bulky, expensive sterilising systems? Instead, it's time to welcome the Haakaa Silicone Microwave Steriliser Bag, a simple, non-nonsense way to sterilise all your baby's small items. Made from reusable, non-toxic, food-grade silicone, this Microwave Steriliser Bag is a cinch to use. Just put some water in the bottom, pop in your washed items, close all but a 3-5cm vent in the top, then sterilise in the microwave! We've even included a handy guide so you can work out just how long you'll need to set your microwave for. This bag even makes it easier to travel – lightweight and compact, you can take it with you wherever it's required, and all you need is a standard microwave to use it in. Say goodbye to all those harmful germs and fiddly, complicated sterilising solutions and enjoy peace of mind with the Silicone Microwave Steriliser Bag.

### Features:

- Reusable and durable alternative to plastic steam bags.
- Effortlessly sterilise microwave-safe baby essentials such as bottles, dummies, tableware, and more.
- Utilises steam sterilisation to kill germs and bacteria.
- 100% food-grade silicone to withstand extreme temperatures.
- Perfect for on-the-go – features a foldable design for compact storage and an easy-carry handle.
- The broad base ensures a stable, upright position while in use.
- Includes a storage loop so it's convenient to hang and dry.
- Sealing strip and buckle for secure closure, minimising the risk of leakage.
- Clear measurement marks on the bag for precise water filling.
- Fridge, freezer, microwave and dishwasher safe.
- BPA, PVC and phthalate-free.


### Cleaning and Care:

Wash before each use. Although dishwasher safe, we recommend hand washing with warm, soapy water, then rinsing and drying thoroughly. Do not use bleach-based cleaners or tablets to clean this product. Only soft bristle brushes or soft sponges should be used to clean this product, as hard scourers may scratch the surface. Store product in a cool, dry place and avoid direct sunlight.


### Caution:

- For adult use only.
- To avoid suffocation, keep this bag away from children and pets. This bag is not a toy.
- This product is to be used with items that are microwave-safe only. Do not use bag on stovetops, in ovens, or on/under grills.
- Be sure to follow the instructions provided, as heating for longer than recommended may warp the bag's contents. Using less than 100ml of water may also lead to warping of contents.
- Check product condition regularly. Replace if this product shows any signs of damage.
- Avoid placing sharp objects in or on the bag, as they may damage it.
- Follow manufacturer's instructions on the use of your microwave.


### Instructions:

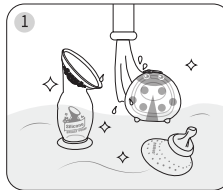


Clean the microwave-safe items to be sterilised in warm, soapy water and rinse thoroughly.

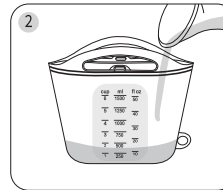

Add a minimum of 100ml of clean water to bag. Water should not cover items.

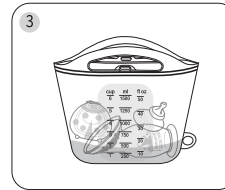

Place items into the bag, making sure it isn't overfilled.

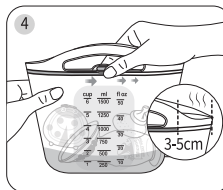


Close the sealing strip and fasten the buckle. Make sure there is a 3-5cm gap for steam to escape.

**NOTE:** During sterilisation, seal may open. This is normal. Ensure buckle is securely closed to keep contents safe.



With the bag standing upright, microwave on high power in the centre of the microwave, following the provided timing guide.

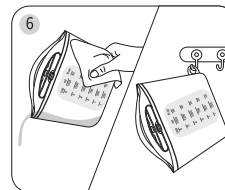

After microwaving, allow bag to cool to room temperature. Carefully remove bag from the microwave, then remove the sterilised items. Let the bag dry completely before storing.

### Microwave times

500 - 720W	7 minutes
800 - 1100W	4 minutes
1100W +	3 minutes

\*Be sure to follow timing instructions as heating longer than recommended may distort the shape of the items inside or cause damage. If unsure about your microwave's wattage, check with your manufacturer.

### WARNING:

Never completely close bag when sterilising, as steam may build up inside and cause bag to pop open. Always leave recommended 3-5cm gap for steam to escape. Use care when removing from microwave as bag will be filled with hot water and steam after use, which may cause burns. Please hold bag from the other side of the vent gap when removing from the microwave to avoid escaping steam. Use a towel or oven mitts when removing bag from microwave. DO NOT tip bag when removing from microwave.